

Evodial

DESIGNED FOR:

HFHD (High flux)

OTHER APPLICABLE THERAPIES:

CONVECTIVE (HDF-HF, AFB-K)

MEMBRANE:

HEPRAN (heparin-grafted **AN69 ST**, BPA-free)

SPECIALIZED FOR HIGH BLEEDING RISK PATIENTS

The **Evodial*** dialyzer series is specialized for patients with a high risk of bleeding^{1,2}. It has been designed with the **HeprAN** heparin-grafted membrane^{3,4} and provides a convenient solution for patients requiring reduced or even heparin-free dialysis¹.

FOCUSED ON HEPARIN-FREE DIALYSIS

- May increase the rate of successful heparin-free HD therapy sessions, compared to the current standard of care for high bleeding risk patients¹
- May allow reduced systemic heparin dosing, without compromising the dialysis sessions^{4,5,6}
- Study data indicate that no significant amount of heparin is released from the membrane during a dialysis session⁷

WITH ENHANCED CONVENIENCE¹

- May reduce nurse workload and disposable consumption
- This could result in a lower use of healthcare resources, compared to standard heparin-free dialysis
- Polyvalent dialyzer design, which can accommodate standard hemodialysis, but also convective therapies (hemodiafiltration and hemofiltration), as well as acetate-free-biofiltration (AFB-K, with potassium profile)



The images are for illustration purposes only and may differ from the actual product.

*** Do not use **Evodial** in patients with a known allergy to heparin or type II thrombocytopenia caused by heparin (HIT syndrome type II)**

Evodial Specifications

MATERIALS	EVODIAL 1.0	EVODIAL 1.3	EVODIAL 1.6	EVODIAL 2.2
Membrane	HeprAN (heparin-grafted AN69 ST) Acrylonitrile and Sodium methallyl sulfonate blend BPA-free			
Potting	Polyurethane (PUR)			
Housing	Polycarbonate (PC)			
Surface treatment agent	Polyethyleneimine (PEI)			
Protection caps	Polyethylene (PE): Blood caps (HDPE)/Dialysate caps (LDPE)			
Sterilization	Gamma ray [wet]			
Sterile barrier	PET/Aluminium/LDPE			
SPECIFICATIONS				
UF-Coefficient (mL/h*mmHg)*	30	40	50	65
KoA urea*	530	637	824	1045
Blood Compartment volume (mL)	66	83	100	129
Minimum recommended priming volume (mL)	1000 [at UFR = 2000 mL/h]			
Maximum TMP (mmHg)	450			
Recommended Q _B (mL/min)	150-400	200-400	200-500	200-500
Storage conditions	≥4°C (or ≥39°F) and ≤30°C (or ≤86°F)			
Units per box	24			
Gross/net weight (g)	216/188	233/205	284/251	327/295

MEMBRANE

Effective Membrane Area (m ²)	1.05	1.30	1.65	2.15
Fiber inner diameter (µm)	210			
Fiber wall thickness (µm)	45.5			

SIEVING COEFFICIENTS

Vitamin B12 (1,4 kDa)	1.0			
Inulin (5,2 kDa)	0.96			
Myoglobin (17 kDa)**	0.7			
Albumin (66,4 kDa)**	<0.0065			

* According to ISO 8637-1

- UF-Coefficient: measured with bovine blood, Hct 32%, Pct 60g/L, at 37°C

- KoA urea: calculated at Q_B=300 mL/min, Q_D=500mL/min, UF=0 mL/min

- Sieving coefficients: measured with bovine (or human**) plasma, Q_B=300 mL/min, UF=60 mL/min

- Clearances In-Vitro: measured at UF=0 mL/min, ±10% [excepted for vit.B12 ±20%]

1. Laville M, et al. *Results of the HepZero study*. *Kidney Int* 2014; 86:1260-1267.
2. Kessler M, et al. *Anticoagulation in chronic hemodialysis: progress toward an optimal approach*. *Semin Dial* 2015; 28:474-489.
3. Thomas M, et al. *AN69: Evolution of the world's first high permeability membrane AN69: Evolution of the world's first high permeability membrane*. *Contrib Nephrol* 2011; 173:119-129.
4. Kessler M, et al. *Heparin-grafted dialysis membrane allows minimal systemic anticoagulation in regular hemodialysis patients: A prospective proof-of-concept study*. *Hemodial Int* 2013; 17:282-293.
5. Morena M, et al. *Biocompatibility of heparin-grafted hemodialysis membranes: Impact on monocyte chemoattractant protein-1 circulating level and oxidative status*. *Hemodialysis International* 2010; 14:403-410.
6. Frascá GM, et al. *Post-Dilution Hemodiafiltration With a Heparin-Grafted Polyacrylonitrile Membrane*. *Ther Apher Dial* 2015; 19:154-161.
7. Baxter. Data on File. *Evodial Heparin leaching data*. Study report BM10-008.

The hemodialyzer/filter is intended for use in hemodialysis, hemodiafiltration and hemofiltration for the treatment of chronic or acute renal failure.

The products meet the applicable provisions of Annex I (Essential Requirements) and Annex II (Full quality assurance system of the Council Directive 93/42/EEC of 14 June 1993, amended by Directive 2007/47/EC)

For safe and proper use of the device, please refer to the Instructions for Use

CE 2797

Baxter, AN69, Evodial and HeprAN are trademarks of Baxter International Inc. or its subsidiaries.

GBU-RC20-200001 - December 2020

MANUFACTURER
Gambro Industries
7, Avenue Lionel Terray
69883 Meyzieu Cedex
France

Baxter Healthcare Corporation
One Baxter Parkway
Deerfield, IL 60015
USA
1-800-422-9837

CLEARANCES IN VITRO (mL/min)*	EVODIAL 1.0	EVODIAL 1.3	EVODIAL 1.6	EVODIAL 2.2
Urea (60 Da) (Q _B -Q _D , mL/min)				
200/500	173	181	189	195
300/500	216	231	250	265
400/500	241	261	287	310
500/500			311	338
Creatinine (113 Da)				
200/500	156	166	176	184
300/500	187	204	220	237
400/500	205	226	246	269
500/500			263	290
Phosphate (142 Da)				
200/500	135	146	156	168
300/500	156	172	187	207
400/500	168	187	205	230
500/500			216	244
Vitamin B12 (1.4 kDa)				
200/500	85	96	111	126
300/500	92	106	124	143
400/500	96	111	131	153
500/500			136	159